


Solution Hybrid Quick Release



Basket

With handle & mount brackets.

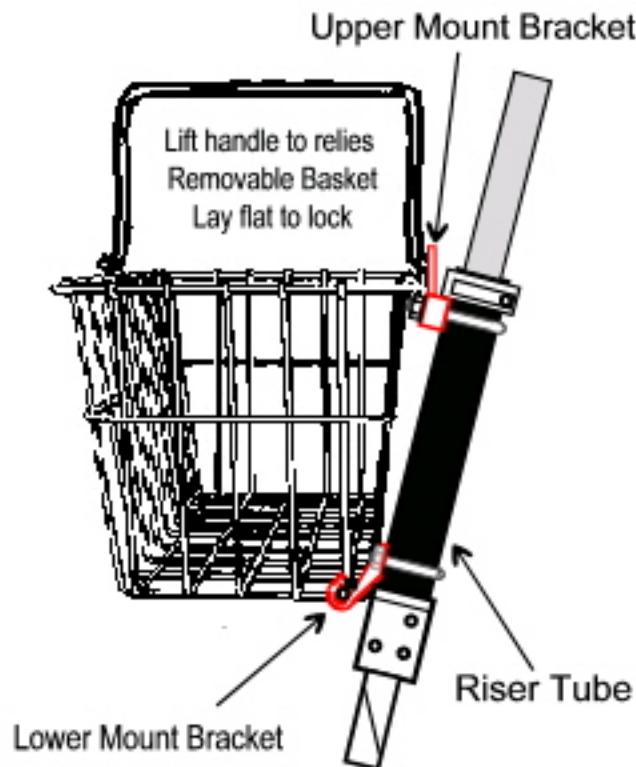

Assembly Instructions

- 1 Place the basket bottom mount at the bottom of the handlebar riser tube with the hooks and openings facing forward and up.
- 2 Slide the U Bolt around the riser tube from the back and through the mount bracket.
- 3 Place the reinforcing plate over the threaded portion of the U Bolt.
- 4 Retain the assembly with the nuts provided.
- 5 Square the bracket and tighten the nuts.
- 6 Place the top mount bracket near the top of the riser tube with the slotted portion facing forward and pointing up.
- 7 Slide the U Bolt around the riser tube from the back and through the mount.
- 8 Snug the mount with the bolts provided.
- 9 Place the basket on the lower mount bracket and swing the handle latch into the upper mount with the latch at the uppermost portion of the slot and lower the handle.
- 10 Tighten the bolts with the basket in place.

Solution Hybrid LLC
21245 Coachman Ave.
Port Charlotte, Fl. 33952
www.solutionhybrid.com

